



User Guide: Create XR

VR App for 3D Drawing

What is Create XR?

CreateXR is a 3D drawing application in virtual reality. It is designed to support relaxation, self-expression, and the development of emotional and cognitive skills.

Tools Panel

On one of your controllers you will find the tools panel. Move the thumb button right/left to scroll between the screens.



Brushes Menu





Explore different brushes to enrich your drawing with different textures.



Color Picker Menu

Change the color to your liking.

Tools Menu

-  **Feet:** teleport yourself to a different spot around you and observe your drawing from a different angle.
-  **Arrows:** undo/redo your last action.
-  **Eraser:** erase manually your drawing.
-  **Camera:** take a picture of your drawing.



Exiting the Environment

- Point at the book icon that appears on the bottom of the virtual controller, to display the options menu
- Click on the door icon to exit the experience



Switch Controller Functions

- Gently tap the bottom of the two controllers together to switch the roles of their functions.

Scale

- Press the Grip buttons on both of the controllers at the same time to adjust the scale of your drawing.
- Use this function to do a more detailed drawing.

