






## Quick Start Guide:

### How to use Meta Casting

#### Temporary Workaround for Casting Issues


#### 1. On the Headset:

- From the Launcher screen, click the casting icon.
- Click Use Meta Casting (note: the 6-digit code appearing here is not the code you need).
- From the device list, select Web, click Next, and the 6-digit code will appear

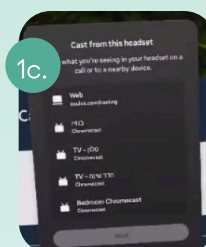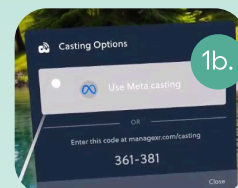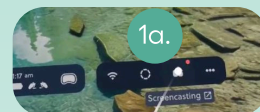


#### 2. On your computer:

- Go to: <https://www.oculus.com/casting/xrhealth343818>  
You can also access this page directly from the Clinician Platform – either from the login page or from the Devices list.
- Enter the 6-digit code shown on the headset.

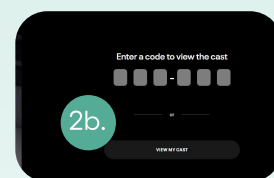

#### 3. To Stop Casting:

- On the headset, go back to the Launcher screen and click the casting icon again.
- Select "Use Meta Casting".
- Click "Stop Casting".

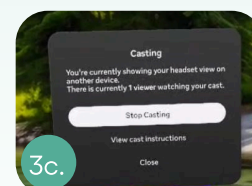

**Note:** There might be problems if the headset and computer are on different networks. If so, you can try asking the patient to cast to their own computer and then share their screen with the clinician.